

PRODUCT INSERT **TILLER HEIGHT AND CONSOLE ANGLE ADJUSTMENTS**

NOTE: The following information, which explains the tiller height and console angle adjustments, is in addition to the information contained in the PMV Series Owner's Manual provided with your personal mobility vehicle. Please retain this information with the PMV owner's manual and contact your authorized Provider with any questions.

TILLER HEIGHT ADJUSTMENT

The height of the tiller console can be adjusted up or down to a comfortable driving level.



WARNING! This procedure should only be performed by an authorized Provider.

To adjust the tiller height:

1. Insert a 5 mm hex key into the opening on the side of the tiller and loosen the bolt. **See figure 1.**
2. Move the tiller up or down to a comfortable position.
3. Tighten the bolt.

CONSOLE ANGLE ADJUSTMENT

The console can be adjusted to a comfortable driving angle.



WARNING! This procedure should only be performed by an authorized Provider.

To adjust the console angle:

1. Insert a 6 mm hex key into the opening on the side of the tiller and loosen the bolt. **See figure 1.**
2. Adjust the tiller to a comfortable driving position.
3. Tighten the bolt.

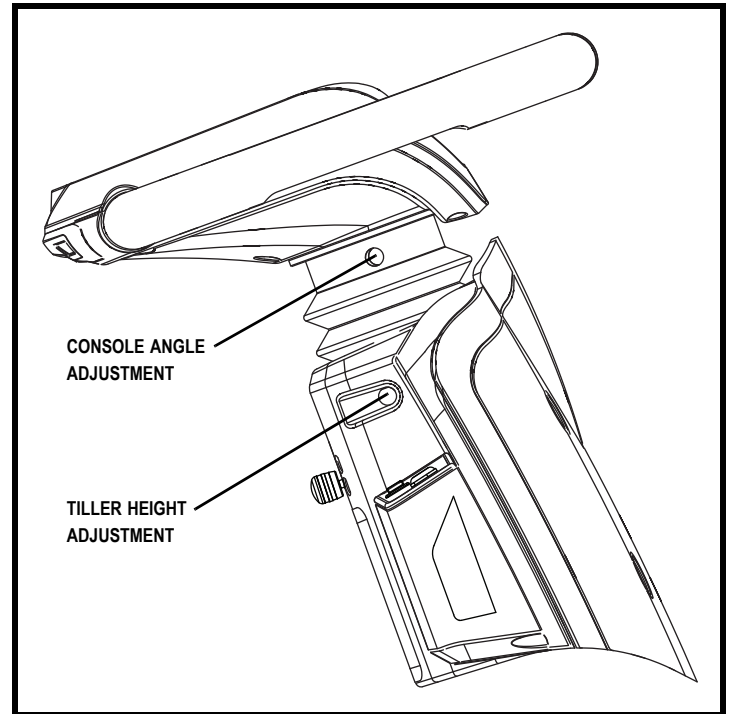


Figure 1. Tiller Height/Console Angle Adjustments

