This product insert provides additional seat height adjustment and seat suspension adjustment instructions to the Victory 9, Victory 10, and Victory XL 130 scooter owner's manuals to reflect the new comfort seat post with suspension. You must read this document in its entirety. In addition, you must read all instructions, warnings, and notes contained in your scooter owner's manual before initial operation. Please keep this document with the scooter owner's manual, and contact your authorized Pride Provider with any questions.

SEAT HEIGHT ADJUSTMENT - COMFORT SEAT POST

To reposition the seat to one of two different heights:

1. Remove the seat from your scooter. Push forward and hold the seat rotation lever to disengage the seat, then rotate the seat and lift it off of the scooter.
2. Remove the rear shroud.
3. Push up the seat post boot to uncover the hardware.
4. Push down on the seat post while loosening and removing the shoulder bolt, washer, and nut. See **figure 1**.
5. Remove the comfort spring.
6. To raise the seat to the topmost mounting hole, insert the height adjustment spacer. To lower the seat to the lowest mounting hole, remove the height adjustment spacer.
7. Reinstall the comfort spring.
8. Hold the seat post at the desired height and match up the locating holes in the seat post tower with the seat post slot. Utilize the top holes for the upper position or the bottom holes for the lower position.

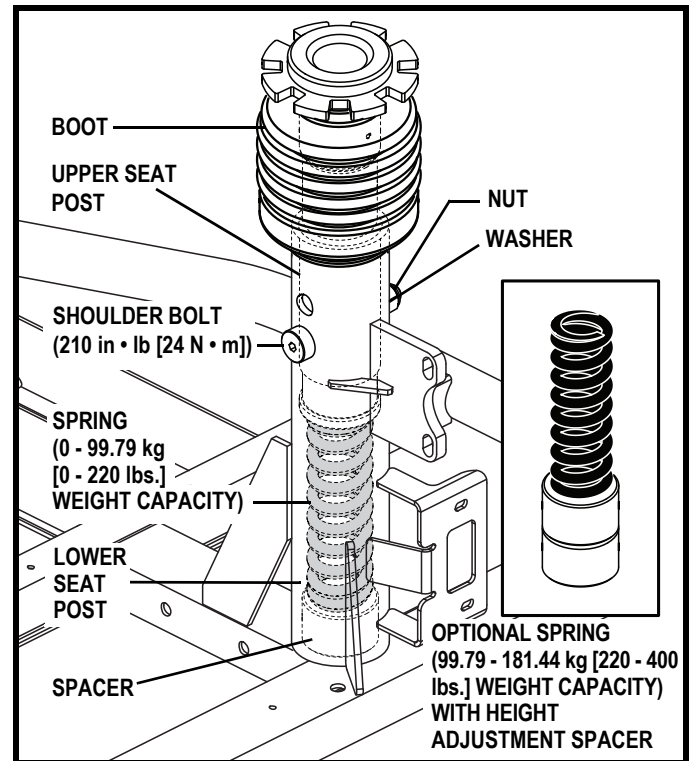
Figure 1. Seat Height Adjustment - Comfort Seat Post

NOTE: When the seat post is mounted in the upper position, you must utilize two spacers.

9. Press down on the seat post while reinstalling the hardware and tightening to 210 in • lb (24 N • m).
10. Replace the rear shroud and the seat.

SEAT SUSPENSION ADJUSTMENT

You can loosen or stiffen the seat suspension based on which comfort spring you install.

To adjust the seat suspension:

1. Remove the seat from your scooter. Push forward and hold the seat rotation lever to disengage the seat, then rotate the seat and lift it off of the scooter.
2. Remove the rear shroud.
3. Push up the seat post boot to uncover the hardware.
4. Push down on the seat post while loosening and removing the shoulder bolt, washer, and nut.
5. Remove the current spring.
6. Install the desired spring. See **figure 1**.
7. Press down on the seat post while reinstalling the hardware and tightening to 210 in • lb (24 N • m).
8. Replace the rear shroud and the seat.

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